- STEP 1:** Select site.
- STEP 2:** Wash site using a plain soap, dry, and then clean with alcohol wipe. Allow site to air dry.
- STEP 3:** Peel lid completely off Sensor Pack. Unscrew cap from Sensor Applicator.

CAUTION: Sensor codes must match on Sensor Pack and Sensor Applicator. Do NOT use if damaged or already opened.

- STEP 4:** Line up dark marks on Sensor Applicator and Sensor Pack. On a hard surface, press down firmly on Sensor Applicator until it comes to a stop.
- STEP 5:** Lift Sensor Applicator out of Sensor Pack.
- STEP 6:** Sensor Applicator is ready to apply Sensor.

CAUTION: Sensor Applicator now contains a needle. Do not touch inside Sensor Applicator or put it back into Sensor Pack.

- STEP 7:** Place Sensor Applicator over site. Push down firmly to apply Sensor.
- CAUTION:** Do NOT push down on Sensor Applicator until placed over prepared site to prevent unintended results or injury.
- STEP 8:** Gently pull Sensor Applicator away.
- STEP 9:** Make sure Sensor is secure. Discard the used Sensor Pack and Sensor Applicator. See *Disposal* section in the User's Manual.

To use your Sensor, refer to all the labeling provided with the Reader Kit. If using FreeStyle LibreLink, refer to the User's Manual in the App.

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
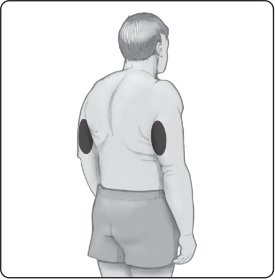
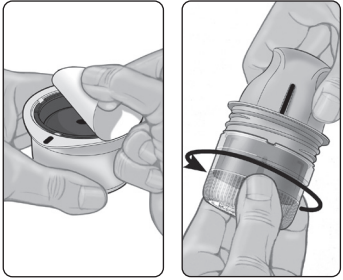
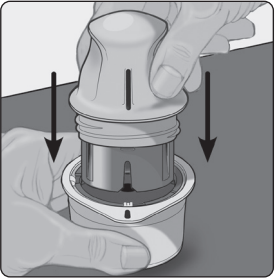
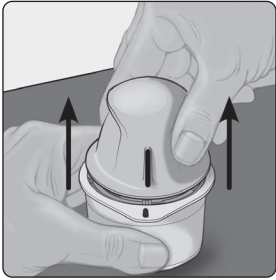
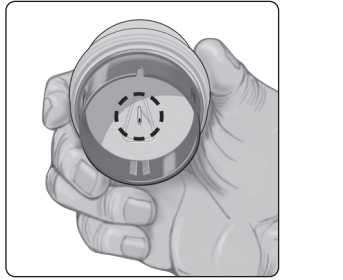
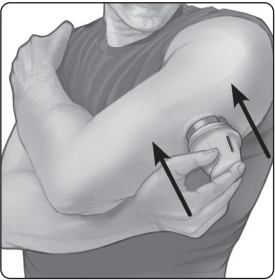
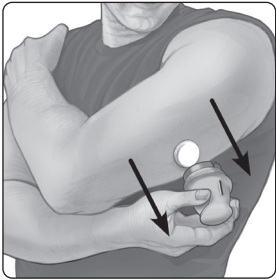
9

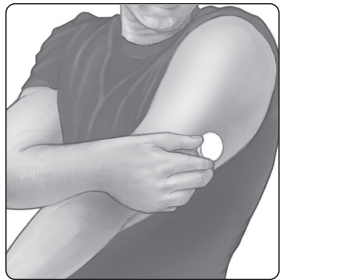FreeStyle Libre Sensor

READ ALL THE LABELING PROVIDED WITH THE READER KIT BEFORE HANDLING THE SENSOR KIT PARTS. IF USING FREESTYLE LIBRELINK, REFER TO THE USER'S MANUAL IN THE APP.

For use with: FreeStyle Libre Reader and FreeStyle LibreLink app. (Not all products are available in all countries.)

Indications for Use

The FreeStyle Libre Flash Glucose Monitoring System, when used with a compatible device, is indicated for measuring the glucose level in the interstitial fluid of people (4 years of age and older) with or without diabetes mellitus, including pregnant women. The sensor and compatible device are designed to replace blood glucose testing in diabetes self-monitoring, including insulin dosing. The indication for children (between 4 and 12 years old) is restricted to those who are supervised by a caregiver who is at least 18 years old. The caregiver is responsible for handling or assisting the child to operate the sensor and compatible device, and also for interpreting or assisting the minor in interpreting the sensor glucose readings.

CONTRAINDICATION:

The Sensor must be removed prior to Magnetic Resonance Imaging (MRI).

WARNING:

The Sensor contains small parts that may be dangerous if swallowed.

Note: For information on what your Sensor glucose reading means, refer to the User's Manual of the device that you are using with your Sensor.

CAUTION:

- **On rare occasions, you may get inaccurate Sensor glucose readings. If you believe your readings are not correct or are inconsistent with how you feel, perform a blood glucose test on your finger to confirm your glucose and check to make sure your Sensor has not come loose. If the problem continues or if your Sensor is coming loose, remove the current Sensor and apply a new one.**
- Intense exercise may cause your Sensor to loosen due to sweat or movement of the Sensor. If your Sensor comes loose, you may get no readings or unreliable readings, which may not match how you feel. Follow the instructions to select an appropriate application site.
- Do not reuse Sensors. The Sensor and Sensor Applicator are designed for single use. Reuse may result in no glucose readings and infection. Not suitable for re-sterilization. Further exposure to irradiation may cause inaccurate results.
- **The Sensor Pack and Sensor Applicator are packaged as a set and have the same Sensor code. Check that the Sensor codes match before using your Sensor Pack and Sensor Applicator. Sensor Packs and Sensor Applicators with the same Sensor code should be used together or your Sensor glucose readings may be incorrect.**
- Some individuals may be sensitive to the adhesive that keeps the Sensor attached to the skin. If you notice significant skin irritation around or under your Sensor, remove the Sensor and stop using the Sensor. Contact your health care professional before continuing to use the Sensor.

Description

The Sensor initially comes in two parts: one part is in the Sensor Pack and the other part is in the Sensor Applicator. Follow the instructions to prepare and apply the Sensor on the back of your upper arm. The Sensor has a small, flexible tip that is inserted just under the skin. **The Sensor can be worn for up to 14 days.**

Sensor Kit Parts

Sensor Pack
Used with Sensor Applicator to prepare Sensor for use.

Sensor Applicator
Applies Sensor to body.

Alcohol Wipe
Used to prepare skin before Sensor application.

User Safety, Storage, and Handling

- The Sensor Pack is sterile unless opened or damaged.
- Store the Sensor Kit between 4°C-25°C. While you don't need to keep your Sensor Kit in a refrigerator, you can as long as the refrigerator is between 4°C-25°C.
- **Your Sensor has been tested to withstand immersion into one meter (3 ft) of water for up to 30 minutes. It is also protected against insertion of objects > 12mm diameter. (IP27)**
- Do not freeze. Do not use if expiration date has passed.
- The alcohol wipe is labeled as follows: reliect pre-injection swab | 1 swab | Use for skin cleansing prior to injection | Saturated with Isopropyl Alcohol 70%. Directions: Tear the sachet open and remove the swab. From the site of injection, briskly rub the skin in a circular motion, moving outward. External use only. Single use only. Discard after use. NEVER use to disinfect an invasive medical device. Caution: Keep away from children. Keep away from eyes. If a swab makes contact, rinse with water. In the event of skin irritation or rash, discontinue use. Do not use after expiration date. Do not reuse.

Disposal

Refer to User's Manual for disposal instructions.

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Abbott Laboratories de Colombia S.A.S.